

Milliken®

AGILE

Fibril Dressings with AFM® Technology

PRODUCT OVERVIEW

Milliken™



AGILE dressings were developed to meet the unmet needs of patients and clinicians for a highly absorbent gellable dressing with increased structural integrity and conformability. AGILE dressings combine the patented Active Fluid Management technology with an innovative weave of gelling fibrils. AGILE has the unique ability to pull excess moisture away from the wound bed, absorbs it into the gelling fibrils or transfers it to a secondary dressing. AGILE is able to repeatedly manage moisture fluctuation during the entire application.




Improving Patient Lives. Together.

THE AGILE DIFFERENCE

Trusted Moisture Management

Builds on Milliken's knowledge and expertise of moisture managing textiles and can be trusted to be superior

- Is highly absorbent
- Uses gellable fiber construction
- Promotes healing through a moist wound environment
- Protects periwound from harmful exudate (reduces risk of maceration)
- Removes harmful exudate from the wound

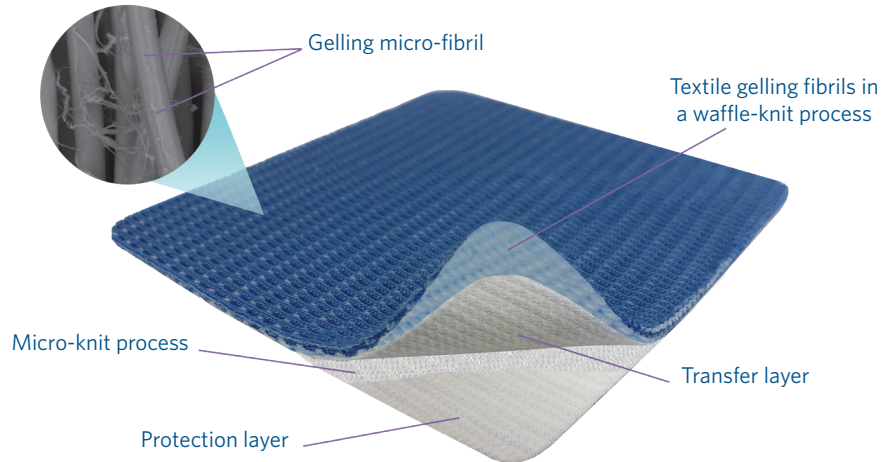

Versatility of Use


Available in a variety of shapes and sizes providing the ability to be used on many different wounds throughout their healing stages

- Able to be cut
- Maintains structural integrity even when wet
- Can be worn up to seven days
- Offers conformability
- Provides low adherency

Comfort of Mind

Brings comfort to both clinicians and patients by incorporating Milliken's unique approach to moisture management


- Increases caregiver confidence in performance
- Delivers patient comfort during application
- Offers low adherency and high breathability

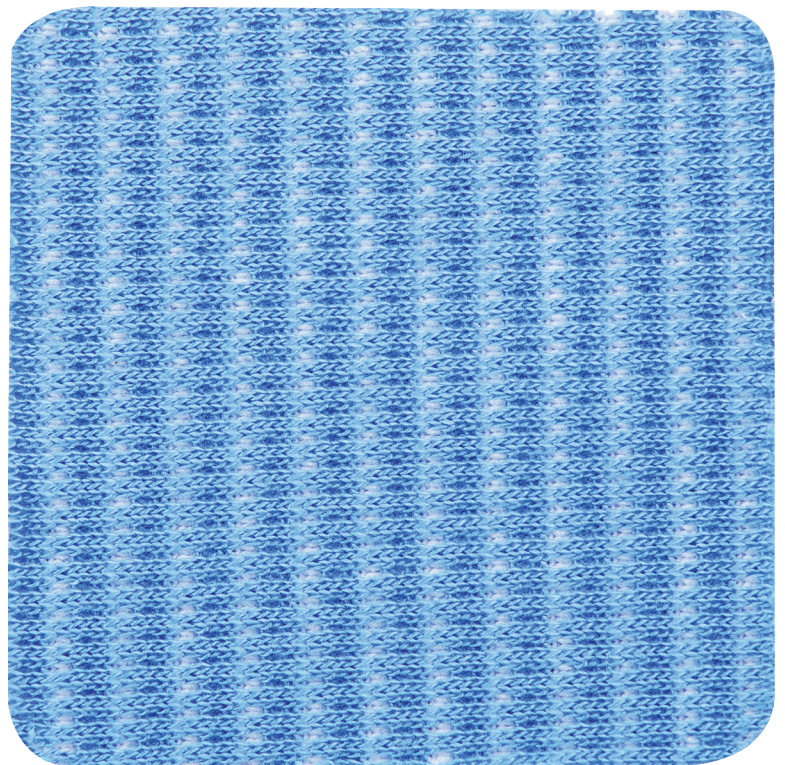

Actual size 4" x 4"


AGILE DRESSING APPLICATION

1

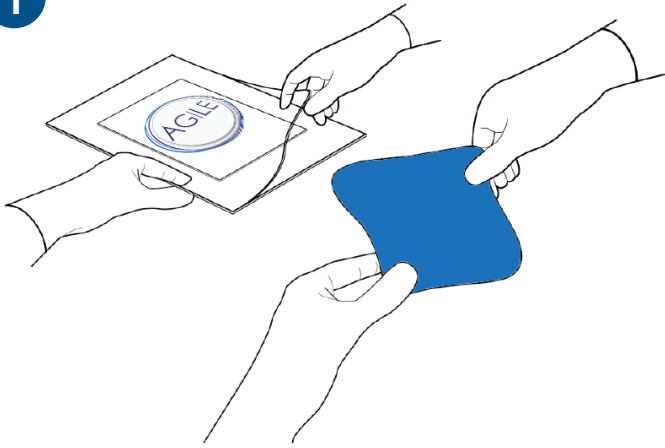


Remove AGILE dressing from package. If required, dressing can be cut to size.

2

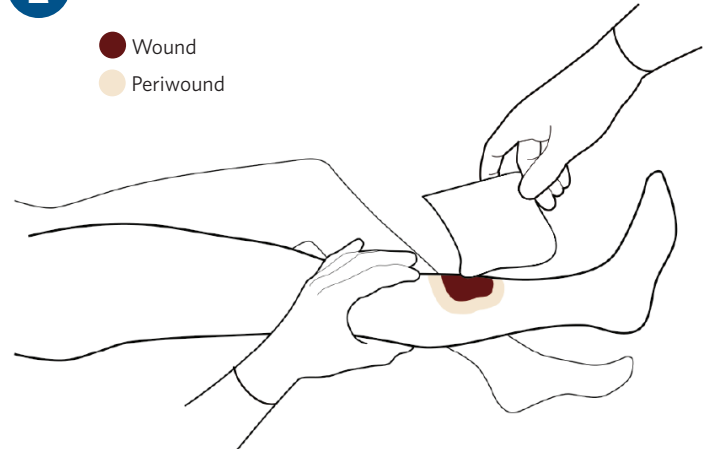


Apply AGILE with blue side of dressing away from the wound.

3

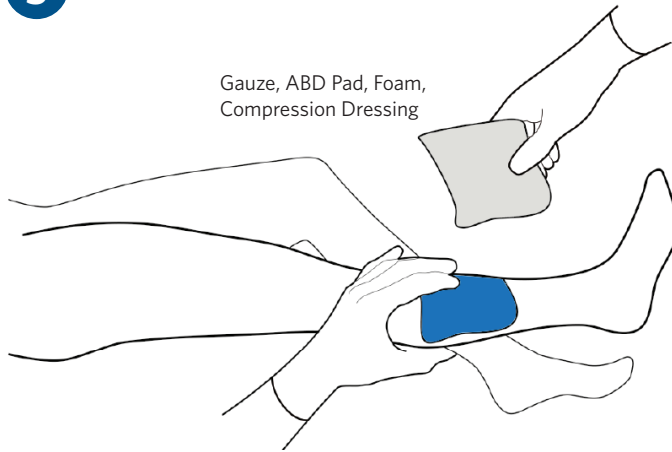

Add absorbent secondary dressing on top of the AGILE dressing (eg gauze, ABD pad, foam, compression dressing) if needed.

4

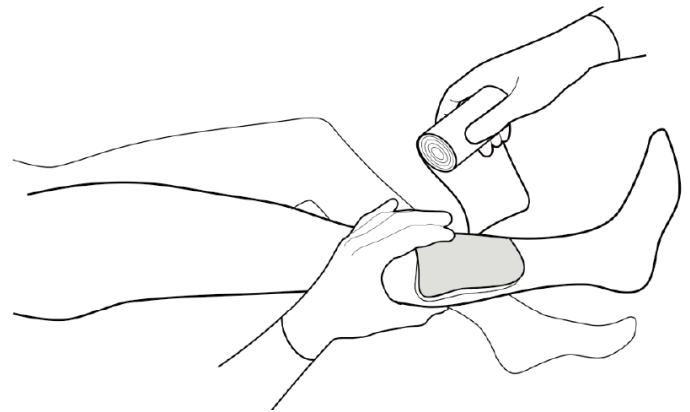

Secure using standard clinical practices.

AGILE		Size	Package	HCPCS	AGILE		Size	Package	HCPCS
3000119991		2" x 2"	10 ea./box 5 bx/cs.	A6196	3000119992		4" x 4" Pre-Cut	10 ea./box 5 bx/cs.	A6196
3000117222		4" x 4"	10 ea./box 5 bx/cs.	A6196	3000119993		4" x 10"	2 ea./box 5 bx/cs.	A6197