


All-in-One wound management system:



PetFlex®·AFD ABSORBENT FOAM DRESSING

- Foam pad and PetFlex cohesive in one
- Quick and easy to apply
- Absorbs the equivalent of 29 gauze pads
- Controlled compression - will not constrict
- EasyTear® technology - no scissors needed
- Effective on puncture wounds, abrasions and lacerations
- Conforms to wound bed - reduces maceration and promotes healing
- Won't stick to wound bed

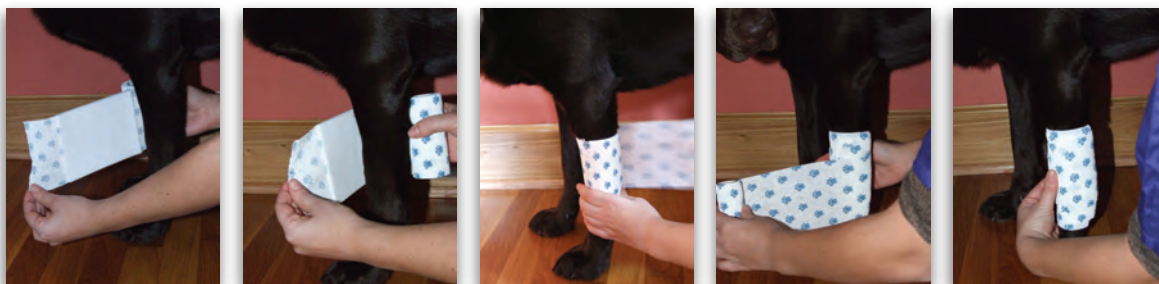

After cleaning the wound, unroll to expose absorbent foam pad.

Place foam pad over wound.

Cover wound with foam pad & wrap bandage around so it covers first layer. with 50% overlap. Prior to each wrap around leg, pull bandage out approx. 6".

Continue to wrap at desired tension

Seal end of bandage to bottom layer with finger pressure for a strong, solid seal.

PetFlex AFD Construction



- 1 PetFlex Layer - Strong & flexible, sticks to itself for easy wrapping
- 2 Waterproof Film - Prevents leakage and contamination
- 3 Absorbent Foam - Patented hydrophillic foam pad draws and holds fluid



PetFlex®·AFD Ordering Information...

Catalog #	Size	Rolls/Case
2220PW-016	2" x 2.5 yds.	16
2440PW-008	4" x 2.5 yds.	8