

Instructions for Use (Read carefully before using the device)

Descriptions

Every Milliken ULTRA Silver foam dressing utilizes a controlled release silver ion coupled with the benefits of Active Fluid Management® (AFM®) technology created by Milliken Healthcare Products, LLC. The three part design combines two layers of high performance fabric using a micro-knit process, creating a powerful moisture transfer mechanism that pulls away and traps excess exudate in the highly absorbent foam layer.

ULTRA Silver combines the moisture management benefits of AFM's three-part design with a proprietary ceramic silver ion technology. ULTRA Silver dressings kill bacteria within the dressing. The dressing demonstrates in vitro activity against microbial strains that can be detrimental to wound healing such as, but not limited to: *S. aureus* (MRSA), *S. epidermidis* (MRSE), *P. aeruginosa*, *K. pneumoniae*, *A. baumannii*.

The ULTRA Silver Dressings provide an antimicrobial barrier to microbial colonization in the dressing and reduce microbial penetration through the dressing.

* Data on file

Directions for Use

ULTRA Silver dressings can be used throughout the healing process. Prepare the wound bed according to clinical protocol such as debridement and/or cleaning and rinsing with sterile saline or water. Remove the dressing from the package and apply to the wound with the printed side of the dressing away from the wound. Optimal performance of ULTRA Silver dressings requires direct contact with the wound. ULTRA Silver dressings should not be folded. If required, ULTRA Silver dressings can be cut to various shapes and sizes. ULTRA Silver dressings can be secured with an appropriate dressing such as roll gauze, adhesive border gauze, composite island dressing, transparent film or tape.

Directions for Removal

ULTRA Silver dressings should be changed when clinically indicated or as prescribed by a healthcare professional. Carefully remove the fixative and the dressing from the wound. Normally the dressing does not adhere to the wound. If adherence is observed, the dressing can be moistened with sterile saline or water to ease removal.



MILLIKEN HEALTHCARE PRODUCTS, LLC

The Milliken logo is a stylized, cursive script of the word "Milliken" in a dark blue color. A small trademark symbol (TM) is located to the left of the first letter.

920 Milliken Road
Spartanburg, SC 29303 USA
Tel: 866-491-6556
8 am - 5 pm E.S.T.

healthcare.milliken.com
medproducts@milliken.com

Made in U.S.A.
© 2019 Milliken Healthcare Products, LLC. All rights reserved. Active Fluid Management® and AFM®
are trademarks of Milliken Healthcare Products, LLC.
Document #3000051018PP15

Indications

The ULTRA Silver dressings are indicated for management of partial thickness burns, incisions, skin grafts, donor sites, lacerations, abrasions, and Stage I-IV dermal ulcers (vascular, venous, pressure and diabetic).

Contraindications

The ULTRA Silver dressings should not be used on patients with a known sensitivity to silver. In case of suspected allergic reaction, discontinue use immediately and contact your healthcare provider.

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner (U.S.A. only).

Precautions and Observations

ULTRA Silver dressings are not approved for use directly on full thickness burns. The Active Fluid Management performance of ULTRA Silver dressings may be impaired by use with petrolatum based ointments.

Storage

ULTRA Silver dressings should be stored at temperatures less than 131 °F. Keep dry.

Sterilization

ULTRA Silver dressings are sterile as provided. Do not use ULTRA Silver dressings if the primary package is visibly damaged or broken. Do not resterilize ULTRA Silver dressings. For single use only.