

Instructions for Use (Read carefully before using the device)

Descriptions

Milliken AGILE dressings utilize micro-fibril gelling technology created by Milliken Healthcare Products, LLC. AGILE's unique design moves fluid away from the wound and provides improved absorption properties. Both features help create an environment that supports the healing process.

Directions for Use

AGILE dressings can be used throughout the healing process. Prepare the wound bed according to clinical protocol such as debridement and/or cleaning and rinsing with sterile saline. Remove the dressing from the package and apply to a highly exudating wound with the blue side of the dressing away from the wound. Optimal performance of AGILE dressings requires direct contact with the wound. If required, AGILE dressings can be cut to various shapes and sizes and it is recommended the dressing overlap the wound margins by about one inch. AGILE dressings can be secured with an appropriate secondary dressing such as rolled gauze, adhesive border gauze, composite island dressing, transparent film or tape. AGILE dressings can be used under compression.

Directions for Removal

AGILE dressings should be changed when clinically indicated or as prescribed by a healthcare professional. Carefully remove the dressing. Normally, the AGILE dressing does not adhere to the wound. If the dressing is adhered, it should be moistened with sterile saline to ease removal.



healthcare.milliken.com
medproducts@milliken.com

920 Milliken Road
Spartanburg, SC 29303 USA
Tel: 866-491-6556
8 am - 5 pm E.S.T.

MILLIKEN HEALTHCARE PRODUCTS, LLC

The Milliken logo is a stylized, cursive script of the word "Milliken" in blue. A small "TM" trademark symbol is located at the bottom left of the word.

Made in U.S.A.
© 2020 Milliken Healthcare Products, LLC. All rights reserved.
AGILEPFD0

Indications

AGILE dressings are indicated for the management of acute and chronic wounds: partial thickness burns, incisions, skin grafts, donor sites, lacerations, abrasions, tunneling, and Stage I-IV dermal ulcers (vascular, venous, pressure and diabetic).

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner (U.S.A. only).

Precautions and Observations

AGILE dressings should not be used with petroleum-based ointment since it may impair the Active Fluid Management performance.

If redness or irritation occurs, discontinue use.

Storage

AGILE dressings should be stored at temperatures less than 131°F. Keep dry

Sterilization

AGILE dressings are sterile as provided. Do not use AGILE dressings if the primary package is visibly damaged or broken. Do not resterilize AGILE dressings. For single use only.