COFXELT TLC CALAMINE

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DESCRIPTION

These instructions are for both CoFlex TLC Calamine and CoFlex TLC Calamine Lite.
 هذه التعليمات مصممة لكل من CoFlex TLC Calamine وCoFlex TLC Calamine Lite.
 CoFlex TLC Calamine (Standard & Lite) is a two-step non-latex compression system. Layer 1 is a foam roll with Calamine that provides comfort and absorbs drainage. Layer 2 is a stretch, moisture-resistant compression bandage that provides the proper level of sustained working pressures to promote venous return. This system is designed to work together and provide continuous restorative compression for up to one week. A nylon stocking is included for ease of use and patient comfort.

Indications for Use

CoFlex TLC Calamine (Standard & Lite) is designed to treat venous leg ulcers and other related conditions.
 CoFlex TLC Calamine (Standard) can be used for patients with an ABI greater than or equal to 0.8.
 CoFlex TLC Calamine (Lite) can be used for patients with an ABI greater than or equal to 0.5.

Contraindications

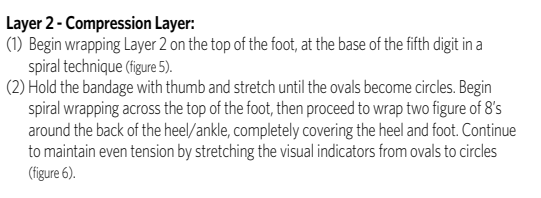
CoFlex TLC Calamine (Standard): ABI less than 0.8.
 CoFlex TLC Calamine (Lite): ABI less than 0.5.

Precautions & Warnings

- Should be used under the care of a licensed healthcare professional.
- Measure the Ankle Brachial Index prior to application.
- CoFlex TLC Calamine (Standard & Lite) Layer 1 is not a primary wound dressing. Follow appropriate protocol for the wound condition before applying.
- CoFlex TLC Calamine (Standard & Lite) Layer 2 may initially be applied with less tension allowing the patient to adjust to or tolerate the short stretch compression system.
- All the discretion of a healthcare professional, patients or their care providers can be trained to apply the bandage.
- Instruct patient to contact physician or nurse if they experience pain, numbness, tingling, discoloration or if swelling occurs.
- Product is not made with latex.
- Single-use item. If reused, indirect restorative compression cannot be assured and increases potential for infection.
- Layer 1 of this product contains talc.
- Redness or irritation occurs, discontinue use.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Layer 2 - Compression Layer:
 1) Begin wrapping Layer 2 on the top of the foot, at the base of the fifth digit in a spiral technique (Figure 5).

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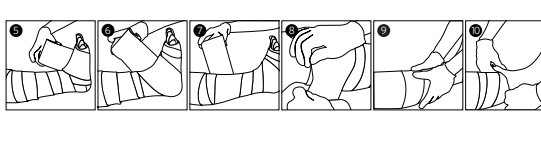
Visual indicators support application and intended compression

Apply stretch bandage until the oval indicators become symmetrical circles.

1) Continue to spiral wrap up the leg, covering it with 50% overlap while maintaining even tension (Figure 7).

2) Cut or hand-trim excess material and tape to keep the bandage in place (Figure 6).

Removal/Disposal

- CoFlex TLC Calamine (Standard & Lite) may be removed by unwrapping or with bandage scissors.
- Dispose of product per your facilities protocol.

تجزین
 For best results, store CoFlex TLC Calamine (Standard & Lite) in its original packaging and use within the expiration date. Avoid excessive heat and humidity. Refer to the label on each box for expiration date.

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